

Accessory Kit



The Contura MLB is sold separately.



Contura™ Accessory Kit

WINDSTONE 008840

Contura Accessory Kit Recommended Use

1. Open Accessory Kit and lay out sterile contents on 30 x 30 white wrap
2. Place 24 x 24 Underpads beneath patient
3. Determine catheter insertion site and prep the area using ChloroPrep Swabsticks
4. Peel backing off of 12 x 16 sterile drapes and frame sterile field
5. Cover ultrasound probe with sterile sleeve (if applicable)
6. Use ultrasound gel packet to enhance ultrasound image as needed
7. Open Contura package and prep balloon per instructions for use
8. Use saline provided to inflate balloon per Contura instructions for use
9. Secure wound using Steri-Strip tape if needed
10. When the implant is complete, place Bacitracin ointment around catheter site
11. Place 4 x 4 split sponge dressing around catheter exit
12. Cover catheter with 4 x 4 Nu Gauze sponge dressing as needed
13. Secure 4 x 4 gauze dressing using Micropore tape provided

Lidocaine, sterile gloves and needles are not included in this kit but will be necessary for the implant procedure. The Contura MLB is sold separately.

To order Contura™ Accessory Kit
(WINDSTONE 008840)
\$24.50 per Kit

Contact SenoRx Customer Service at 877-210-1300
or customerservice@senorx.com



INNOVATIVE BREAST CARE AT EVERY STEP

11 Columbia
Aliso Viejo, California 92656
877-210-1300
949-362-3519 Fax
customerservice@senorx.com
www.senorx.com

Rx Only

Please consult product Directions For Use for indications, contraindications, warnings, and precautions.

Contura is a trademark of SenoRx, Inc. ChloroPrep is a registered trademark of Enturia, Inc. Micropore and Steri-Strip are registered trademarks of 3M. Nu Gauze is a registered trademark of Johnson & Johnson.

©Copyright 2008 SenoRx® Inc. All rights reserved.